# DiveP

*Beschreibung / Manual*

**HESER**

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<b>1 Deutsche Version</b>	<b>3</b>
1.1 Teileliste . . . . .	3
1.2 Beschreibung . . . . .	3
1.3 Warnhinweise . . . . .	3
1.4 Montage . . . . .	5
1.5 Wartung . . . . .	6
1.6 Bedienung . . . . .	6
1.7 Entsorgung . . . . .	7
1.8 Hersteller . . . . .	7
<b>2 English Version</b>	<b>8</b>
2.1 Parts List . . . . .	8
2.2 Product description and intended use . . . . .	8
2.3 Warnings . . . . .	8
2.4 Assembly . . . . .	10
2.5 Maintenance . . . . .	11
2.6 Workflow . . . . .	11
2.7 Disposal . . . . .	11
2.8 Manufacturer . . . . .	12

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## **2 English Version**

These operating instructions are also to be understood as operating instructions for spare parts and partial products, as they explain the handling in connection with the overall product.

### **2.1 Parts List**

If you have purchased a spare part or partial product, this list only includes a part of your purchased product. It is not the scope of delivery. This list is only to be understood as a parts list.

- 1 x DiveP

### **2.2 Product description and intended use**

The DiveP is used by women to connect a PValve.

### **2.3 Warnings**

These operating instructions do not replace any training and do not qualify you for diving or any other technical work. The relevance of the safety instructions always depends on the degree of difficulty of the dive and its requirements. The same applies to the execution of technical work. These safety instructions can therefore not be regarded as comprehensive or always appropriate.

- This description must always be made available to the user.

- The product may only be used for its intended purpose.
- The training and level of knowledge must fully cover the intended use.
- The user's level of training may restrict the intended use.
- If the user does not understand this description or has doubts, he must not use the article.
- These instructions are not a substitute for training.
- Defective, broken or modified products must not be used.
- The contact of normal steel parts with stainless steel parts also leads to rust stains on stainless steel parts.
- Test the skin adhesive and the cleaning wipes **before** sticking to insensitive areas of skin in advance.
- Test the adhesion of the DiveP on the forearm beforehand.
- All areas to which the DiveP is attached must be well and smoothly shaved, otherwise removing it can be very painful.
- Test the DiveP **before** using it for a dive extensively at home so that you can remove it immediately if you feel unwell.
- Only use adhesives that are approved as skin adhesives to attach the DiveP.
- Only use approved cleaning cloths and agents to clean the skin.
- The DiveP will cause pain when removed. If you have any concerns, only apply the DiveP at one point and test the removal process for yourself first.
- If you are concerned about skin irritation, consult a doctor.

## 2.4 Assembly

- The intimate area should be completely shaved. It is best to shave one or two days beforehand. If the skin has been freshly shaved, it may cause a brief burning sensation. Apply
- skin adhesive to the adhesive surfaces of the DiveP without any gaps. There must be a continuous adhesive film with no gaps.
- Allow the adhesive to air well before applying. The adhesive may already be almost “dry”.
- To stick on the DiveP, press the container together slightly and press carefully all around. Whether you apply the DiveP from the front or the back must be tried out. How you position the DiveP for gluing must be tried out. There must be no skin folds under the DiveP.
- When bonding, the legs should not be spread too far apart, as the legs are not too far apart during the approach to the dive site and when diving. This ensures that the DiveP stays in place during the dive. If you wish, you can put on one or two tight thongs to press the DiveP against the perineum.
- The DiveP is glued on with the hose connection facing forwards.
- For a better adhesive effect, a little silicone skin adhesive can also be applied directly to the perineum or other areas.
- Avoid bringing the liquid adhesive into contact with clothing, as it is difficult or impossible to remove completely.
- As with any skin adhesive, the adhesive effect may not be sufficient in individual cases, so that the DiveP may come off prematurely.
- Effects on the skin, such as redness, can never be completely ruled out when using skin adhesives. Oil-free skin creams can help here.

- After the dive or diving day, remove the DiveP and rinse well with water. It is important that it dries completely in the air. If desired, the DiveP can be cleaned with isopropyl alcohol.
- Please do not store the DiveP in the sun.

It is advisable to test the adhesion of the DiveP a few times at your leisure before the first dive. If the DiveP is not connected to the P-Valve, please empty the container of the DiveP after urinating, as there is residual urine here.

## **2.5 Maintenance**

After every dive in salty water (seawater), all equipment components must be rinsed sufficiently with fresh water.

## **2.6 Workflow**

When using aluminum equipment in seawater, it is essential to avoid contact with stainless steel parts must be avoided. This will lead to contact corrosion and damage the aluminum. The hose connection of the DiveP is connected to the hose of the PValve using the connector of the PValve.

## **2.7 Disposal**

The product must be disposed of at the time of disposal and in accordance with the applicable regional regulations. Electrical appliances must not be disposed of with household waste and must be disposed of in accordance with the applicable regulations. Recyclable materials should always be recycled.

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Heser Tauchtechnik GmbH